

 Women-Friendly

SALIVA OVULATION MICROSCOPE

FHN-002



FEATURES



Portable



Safety and hygiene



Easy to operate



Disposable
Microscope slides*15



App records



Made In Taiwan

Directions

01 Before use

- ♥ Before use, download the "Fore Health" app and register with your personal information and account.
- Gently clean the plastic slides using an eyeglass cleaning cloth or alcohol wipe.

02 During use

- ♥ Place a small amount of saliva on the plastic slides, and wait for 5 to 15 minutes for the saliva to fully dry.
- Then, insert the microscope slides into the "Fore Health Saliva Ovulation Microscope."

03 After use

- ♥ Clip the "Fore Health Saliva Ovulation Microscope" to your smartphone camera (either front or rear) and perform an ovulation pattern check.
- ♥ The test result will display, and you can self-assess and save the record for the day.

